PHILIPS

IMPORTANT INFORMATION (English)	1
INFORMATIONS IMPORTANTES (Français)	7
WICHTIGE INFORMATIONEN (Deutsch)	13
BELANGRIJKE INFORMATIE (Nederlands)	21
INFORMACIÓN IMPORTANTE (Español)	27
INFORMAZIONI IMPORTANTI (Italiano)	35
VIKTIG INFORMATION (Svenska)	41
VIKTIG INFORMASJON (Norsk)	47
VIGTIG INFORMATION (Dansk)	53
重要情報 (日本語)	59

HEARTSTART
DEFIBRILLATORS

This page deliberately blank.
Page intentionnellement laissée vide.
Absichtlich leer gelassen.
Opzettelijk onbedrukt gelaten.
Este espacio se ha dejado en blanco deliberadamente.
Pagina lasciata intenzionalmente vuota.
Sidan är avsiktligen tom.
Skal være tom.
Tom side.
このページは空白です。

IMPORTANT INFORMATION ABOUT THIS HEARTSTART DEFIBRILLATOR, OPTIMIZED FOR GUIDELINES 2005

This HeartStart M5066A or M5067A Defibrillator is optimized for Guidelines 2005,* the latest recommendations on cardiopulmonary resuscitation (CPR) and defibrillator use from the American Heart Association, the European Resuscitation Council, and the International Liaison Committee on Resuscitation. The Guidelines 2005 optimization of the HeartStart Defibrillator consists of changes to configuration parameters, their factory default settings, and voice instructions. This document consists of the sections of the HeartStart's Instructions for Use that are changing as a result of the Guidelines 2005 optimization. It contains small sections of Chapter 3 and Appendix C, and Appendix F in its entirety.

*The information in this document supersedes that contained in the corresponding sections of the Instructions for Use or Owner's Manual shipped with the defibrillator. **Please retain this document for reference.** Philips recommends that medical directors thoroughly understand Guidelines 2005 and how the HeartStart functions, and read the HeartStart's Instructions for Use carefully when planning their response protocol.*

HEARTSTART DIRECTIONS FOR USE (Chapter 3)

The changes to the information in Chapter 3 are limited to Step 3 in using the defibrillator.

STEP 3: PRESS the Shock button. As soon as the HeartStart detects that the pads are attached to the patient, it begins analyzing the patient's heart rhythm. It tells you that no one should be touching the patient, and the Caution light begins flashing as a reminder.

If a shock is not needed, the HeartStart tells you it is safe to touch the patient and instructs you to perform CPR if needed. (If CPR is not needed – for example, if the patient is moving or regaining consciousness – follow your local protocol until emergency medical personnel arrive.) Then, the HeartStart invites you to press the flashing blue i-button for CPR Coaching if desired.

If a shock is needed, the Caution light goes from flashing to solid, the orange Shock button starts flashing, and the defibrillator tells you to press the flashing orange button. When you press the Shock button, the defibrillator tells you that the shock has been delivered. Then the defibrillator tells you it is safe to touch the patient and invites you to press the flashing blue i-button for CPR Coaching if desired.

Treating infants and children. Most cardiac arrests in children are not caused by VF. When responding to cardiac arrest in an infant or child, provide infant/child CPR while a bystander calls EMS and brings the

* 2005 American Heart Association Guidelines for Cardiopulmonary Resuscitation and Emergency Cardiovascular Care. *Circulation* 2005;112:IV.
European Resuscitation Council Guidelines for Resuscitation 2005. *Resuscitation* 2005;67S1:S7-S23.
International Liaison Committee on Resuscitation. *Resuscitation* 2005;67:203-211.

HeartStart. If no bystander is available, provide 1-2 minutes of CPR before calling EMS and retrieving the HeartStart. If you witnessed the child's collapse, call EMS *immediately* and *then* get the HeartStart.

GLOSSARY OF SYMBOLS/CONTROLS (Appendix C)

symbol	description
	Indicates that this HeartStart Defibrillator is optimized for Guidelines 2005. Appears on the HeartStart's rear label.
	Dispose of in accordance with your national or local requirements. By complying with your national regulations regarding disposal of electric, electronic, and battery waste, you can make a positive contribution to our shared environment. Such waste can introduce harmful elements into the environment as a whole and may also endanger human health.

CONFIGURATION (Appendix F)

Device options

The following table includes the features of HeartStart Defibrillator operation that are not related to patient treatment. These have not changed as a result of implementing Guidelines 2005.

parameter	settings	default	default description
speaker volume	1, 2, 3, 4, 5, 6, 7, 8	8	The volume of the HeartStart's speaker is set to 8, highest.
auto send periodic self-test (PST) data	On, Off	On	Enables the periodic self-test data to be broadcast through the device's infrared data port.
ECG out data	On, Off	On	Enables the ECG data to be broadcast through the device's infrared data port.

Patient treatment protocol options

In the following table, there are new default settings for several parameters. These include shock series, protocol pause timer, NSA pause timer, and CPR prompt. There are two new parameters, CPR Coaching adult ventilation instruction and CPR Coaching infant/child ventilation instruction. In addition, the CPR coaching compression:ventilation ratio parameter replaces the CPR prompt rate parameter.

parameter	settings	default	default description
“call EMS” voice reminder timing	<ul style="list-style-type: none"> At power on (when the user turns on the HeartStart) At power on and at the start of the first pause interval At the start of the first pause interval No reminder 	At the start of the first pause interval.	Provides a voice reminder to make sure emergency medical services have been called, at the start of the first pause interval.
shock series	1, 2, 3, 4	1	<p>The automatic protocol pause for CPR is activated each time a shock is delivered.*</p> <p>During the protocol pause, the HeartStart does not perform rhythm analysis.</p> <p>The length of the protocol pause after a shock series is completed is determined by the protocol pause timer setting.</p>
shock series interval (minutes)	1.0, 2.0, ∞ (infinity)	1.0	<p>A delivered shock must occur within 1 minute of the previous shock to be counted as part of the current shock series.</p> <p><i>NOTE: This parameter is only applicable when the shock series is not configured to the default 1 shock.</i></p>

* A shock series begins when a shock is delivered after the HeartStart is turned on. A new shock series begins after a protocol pause. If shock series is configured for 2 or more, a new shock series also begins if the time since the previous shock exceeds the shock series interval setting.

parameter	settings	default	default description
protocol pause timer (minutes)	0.5, 1.0, 1.5, 2.0, 2.5, 3.0	2.0	<p>A 2-minute protocol pause for CPR automatically starts after voice instruction is given when a shock series is completed. After the protocol pause, the defibrillator returns to rhythm analysis.</p> <p>If the user presses the i-button for optional CPR coaching, the HeartStart provides coaching for 5 cycles of CPR, starting and ending with compressions, when the CPR Coaching parameters are also set to their default values. The number of CPR cycles varies for other protocol pause timer and CPR Coaching parameter settings.</p>
NSA pause type	<ul style="list-style-type: none"> Standard NSA pause: HeartStart does not perform rhythm analysis during the NSA pause. SMART NSA pause: HeartStart conducts background monitoring during the SMART NSA pause. If a potentially shockable rhythm is detected, HeartStart terminates the SMART NSA pause and resumes rhythm analysis. 	SMART NSA pause	<p>During a SMART NSA pause, the defibrillator conducts background monitoring. If a potentially shockable rhythm is detected in a motionless patient, the defibrillator terminates the SMART NSA pause and resumes rhythm analysis.</p> <p><i>NOTE: If the HeartStart detects CPR in progress or if the responder has pressed the i-button for CPR Coaching, the SMART NSA pause will be converted to a standard NSA pause. During the standard NSA pause, the defibrillator does not perform rhythm analysis.</i></p>

parameter	settings	default	default description
NSA pause timer (minutes)	0.5, 1.0, 1.5, 2.0, 2.5, 3.0	2.0	A 2-minute NSA pause for CPR automatically starts after voice instruction is given when no shock is advised (NSA).* If the user presses the i-button for optional CPR coaching, the HeartStart provides coaching for 5 cycles of CPR, starting and ending with compressions, when the CPR Coaching parameters are also set to their default values. The number of CPR cycles varies for other NSA pause timer and CPR Coaching parameter settings.
CPR prompt	<ul style="list-style-type: none"> • CPR1: Instructs the user to begin CPR. • CPR2: Instructs the user that it is safe to touch the patient and to begin CPR. • CPR3: Instructs the user to begin CPR and to press the i-button for CPR Coaching. • CPR4: Instructs the user that it is safe to touch the patient, to begin CPR, and to press the i-button for CPR Coaching. 	CPR4: Instructs the user that it is safe to touch the patient, to begin CPR, and to press the i-button for CPR Coaching.	The CPR reminder voice instructions provided at the beginning of a pause interval assures the user that it is safe to touch the patient, instructs the user to begin CPR, and invites the user to press the i-button for guidance in the basic steps of CPR.
CPR Coaching adult ventilation instruction	Yes, No	Yes	Optional CPR Coaching includes rescue breaths at the rate determined by the CPR Coaching compression:ventilation ratio for adults when an adult pads cartridge is installed. <i>NOTE: if this parameter is configured to NO, CPR Coaching will always be compressions-only when an adult pads cartridge is installed.</i>

* If the shock series is configured to 2 or more, and a shock has been delivered as part of a series, the length of the first NSA pause interval within that shock series is determined by the protocol pause timer setting. Otherwise, the length of an NSA pause is determined by the NSA pause timer setting.

parameter	settings	default	default description
CPR Coaching infant/child ventilation instruction	Yes, No	Yes	Optional CPR Coaching includes rescue breaths at the rate determined by the CPR Coaching compression:ventilation ratio for infants and children when an infant/child pads cartridge is installed. <i>NOTE: if this parameter is configured to NO, CPR Coaching will always be compressions-only when an infant/child pads cartridge is installed.</i>
CPR Coaching compression:ventilation ratio	<ul style="list-style-type: none"> • 30:2 adult and 30:2 infant/child • 30:2 adult and 15:2 infant/child • 15:2 adult and 15:2 infant/child 	30:2 adult and 30:2 infant/child	If the user presses the i-button for optional CPR Coaching during a protocol pause or NSA pause, the HeartStart provides coaching in basic CPR for cycles of 30 compressions and 2 ventilations for adults, children, and infants. Pauses begin and end with compressions.

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PHILIPS

Philips Medical Systems is part of
Royal Philips Electronics

Philips Medical Systems

United States

Philips Medical Systems
2301 Fifth Avenue, Suite 200
Seattle, WA, USA 98121
(800) 263-3342

Canada

Philips Medical Systems
281 Hillmount Road
Markham, Ontario
L6C 2S3
(800) 291-6743

Europe, Middle East, and Africa

Philips Medizin Systeme Boeblingen GmbH
Cardiac and Monitoring Systems
Hewlett-Packard Strasse 2
71034 Boeblingen, Germany
(+49) 7031 463-1552

Latin America

Philips Medical Systems
1550 Sawgrass Corporate Parkway, Suite 300
Sunrise, FL 33323, USA
(954) 835-2660

Asia Pacific

Philips Electronics Hong Kong Ltd.
30th Floor, Hopewell Centre
17, Kennedy Road, Wanchai
Hong Kong
(852) 2821 5888

Device manufacturer

The HeartStart Defibrillator is manufactured by Philips
Medical Systems, Seattle, Washington, USA.

013397-0001